



REORDER NUMBER:

RTG-04004

02-Life™



Non-Rebreather Mask

**High Concentration w/ 7' (2.1m) Tubing
750 ml Reservoir Bag Standard Connector**

ADULT

RxOnly

Caution: Federal law (USA) restricts this device to use by or at the direction of a physician

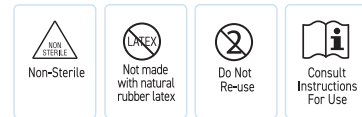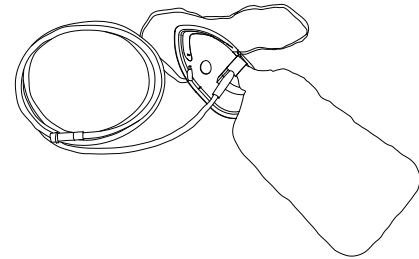

Clean, Ready to Use
For Single Patient Use

INSTRUCTIONS FOR USE:

NOTE:

These directions are general guidelines intended for use by qualified medical personnel.

1. Attach oxygen supply tubing to gas source.
2. Adjust flow as prescribed by physician.
3. Check for flow through the device. Always verify proper function of the valves:
 - The valve between the mask and the reservoir bag should rise on inspiration and lower on exhalation.
 - The valve located on external mask surface should open during exhalation.
4. Place the mask on the patient's face with the elastic strap below the ears and around the neck. Gently pull the strap ends until the mask is secure. Mold the metal strip on the mask to fit the face.

WARNING:

WARNING: Always check for oxygen flow through oxygen inlet valve before placing the mask on the patient.

WARNING: Re-processing of products intended for single use only may result in degraded performance or a loss of functionality.

WARNING: Re-use of single use only products may result in exposure to viral, bacterial, fungal, or prionic pathogens. Validated cleaning and sterilization methods and instructions for reprocessing to original specifications are not available for these products.

WARNING: This product is not designed to be cleaned, disinfected or re-sterilized.

STORAGE TEMPERATURE:
-30°C/-22°F
+40°C/+104°F

For more info

Telephone

| 323-761-0035

E-mail

| info@rtghealth.com

| www.rtghealth.com