



REORDER NUMBER:

**RTG-04016**

**Trach O-Life™**


**Tracheostomy Mask**

**with Swivel Tubing Connector  
and One Side Snap**

**ADULT**

**RxOnly**

Caution: Federal law (USA)  
restricts this device to use by  
or at the direction of a physician

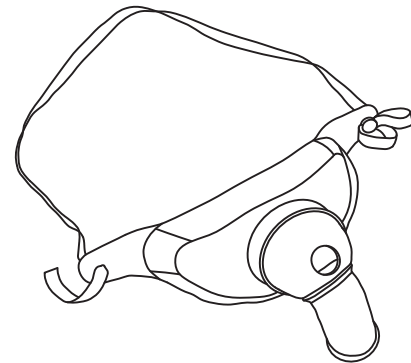

Clean, Ready to Use  
For Single Patient Use

## INSTRUCTIONS FOR USE:

### NOTE:

These directions are general guidelines intended for use by qualified medical personnel.

1. Connect aerosol tubing (not supplied) between the mask and the gas source. The mask inlet swivels 360° to position the tubing for supine or upright patients.
2. Set the gas to the proper flow and check for gas flow through the device.
3. Position the elastic strap behind the neck.
4. Gently pull the end of the strap until the mask is secure.
5. When using suction, loosen the mask and drop the mask out of the work area.

## WARNING:

**WARNING: Re-processing of products intended for single use only may result in degraded performance or a loss of functionality.**

**WARNING: Re-use of single use only products may result in exposure to viral, bacterial, fungal, or prionic pathogens. Validated cleaning and sterilization methods and instructions for reprocessing to original specifications are not available for these products.**

**WARNING: This product is not designed to be cleaned, disinfected or re-sterilized.**

STORAGE TEMPERATURE:  
-30°C/-22°F  
+40°C/+104°F

For more info

 Telephone  
| 323-761-0035

 E-mail  
| info@rtghealth.com

 [www.rtghealth.com](http://www.rtghealth.com)