



REORDER NUMBER:

RTG-07102


Nebu-Life™

Hand Held Nebulizer Cup 6cc

**with Tee Adaptor, 7' Tubing (2.1 m),
and Mouthpiece**

RxOnly

Caution: Federal law (USA)
restricts this device to use by
or at the direction of a physician

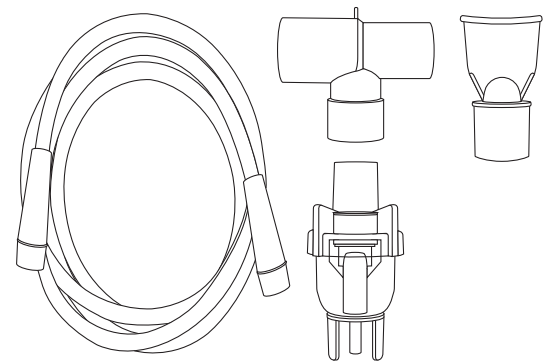

Clean, Ready to Use
For Single Patient Use

INSTRUCTIONS FOR USE:

NOTE:

These directions are general guidelines intended for use by qualified medical personnel.

1. Place prescribed solution into the nebulizer cup.
2. Attach cap to nebulizer cup with a clockwise rotation.
3. Insert nebulizer cup into tee adaptor, aerosol mask or face tent.
4. Connect oxygen tubing to oxygen port of nebulizer cup. Attach the other end of the tubing to gas source.
5. Set gas flow as prescribed by health care professional.
6. Tap the device until it begins to nebulize if necessary.
7. During treatment, periodically tap nebulizer to minimize residual volume.

WARNING:

WARNING: Be sure all connections are secure.

WARNING: Do not attach the outlet of the nebulizer directly to an endotracheal tube, tracheostomy tube or other device that does not offer an alternate pathway for exhalation.

STORAGE TEMPERATURE:
-30°C/-22°F
+40°C/+104°F

For more info

Telephone

| 323-761-0035

E-mail

| info@rtghealth.com

| www.rtghealth.com