



REORDER NUMBER:

RTG-03004



RespVac™

Suction Connecting Tubing

**with Male Connector Non-Conductive Sterile
3/16" X 6'Tube (4.7mm x 1.8m)**

RxOnly

Caution: Federal law (USA)
restricts this device to use by
or at the direction of a physician

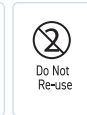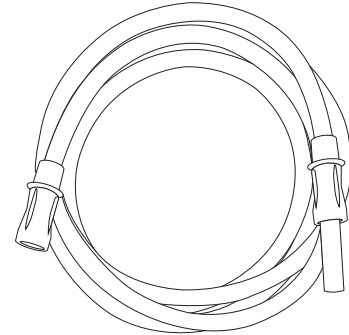

Clean, Ready to Use
For Single Patient Use

INSTRUCTIONS FOR USE:

NOTE: These directions are general guidelines intended for use by qualified medical personnel.

1. Suction connecting tubing is intended to connect a device such as a portable suction machine or an in-wall suction port together with one of the following:
 - a. Suction Catheter
 - b. Yankauer
 - c. Oral Care brush
2. Set the proper suction flow rate as directed by the physician or licensed practitioner.
3. Begin suctioning function on patient.

WARNING:

CAUTION: Be certain that all connections are secure.

WARNING: Re-processing of products intended for *single use only* may result in degraded performance or a loss of functionality.

WARNING: Re-use of *single use only* products may result in exposure to viral, bacterial, fungal, or prionic pathogens. Validated cleaning and sterilization methods and instructions for reprocessing to original specifications are not available for these products.

WARNING: This product is not designed to be cleaned, disinfected or re-sterilized.

STORAGE TEMPERATURE:
-30°C/-22°F
+40°C/+104°F