



REORDER NUMBER:

**RTG-03009**

**RespVac™**

## Suction Connecting Tubing Kit


with Male Connector Non-Conductive Non-Sterile

3/16" x 6' Tube (4.7mm x 1.8m)

3/16" x 18" Tube (4.7mm x 457mm)

### RxOnly

Caution: Federal law (USA)  
restricts this device to use by  
or at the direction of a physician

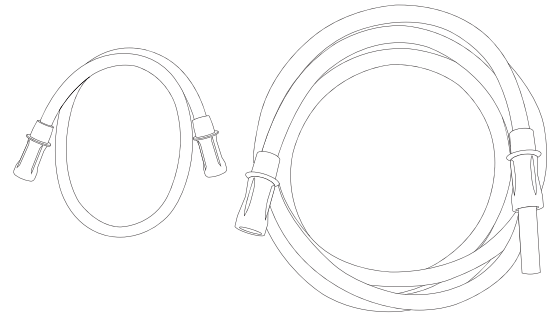

Clean, Ready to Use  
For Single Patient Use

### INSTRUCTIONS FOR USE:

NOTE: These directions are general guidelines intended for use by qualified medical personnel.

1. Suction connecting tubing is intended to connect a device such as a portable suction machine or an in-wall suction port together with one of the following:
  - a. Suction Catheter
  - b. Yankauer
  - c. Oral Care brush
2. Set the proper suction flow rate as directed by the physician or licensed practitioner.
3. Begin suctioning function on patient.

### WARNING:

**CAUTION:** Be certain that all connections are secure.

**WARNING:** Re-processing of products intended for *single use only* may result in degraded performance or a loss of functionality.

**WARNING:** Re-use of *single use only* products may result in exposure to viral, bacterial, fungal, or prionic pathogens. Validated cleaning and sterilization methods and instructions for reprocessing to original specifications are not available for these products.

**WARNING:** This product is not designed to be cleaned, disinfected or re-sterilized.

STORAGE TEMPERATURE:  
-30°C/-22°F  
+40°C/+104°F