



REORDER NUMBER:

RTG-03020


RespVac™

Yankauer Suction Handle

Bulb Tip Non-Vented Sterile

RxOnly

Caution: Federal law (USA) restricts this device to use by or at the direction of a physician

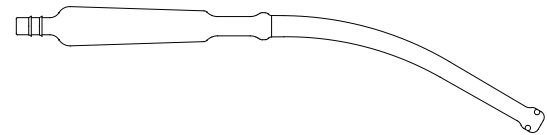

Clean, Ready to Use
For Single Patient Use

INSTRUCTIONS FOR USE:

1. Wash hands with soap and water.
2. Check suction pressure by putting thumb over end of suction tubing.
3. Attach Yankauer to suction tubing.
4. Put on gloves (not included) according to hygiene policy and procedure.
5. Insert Yankauer gently into mouth until it reaches the pouch of the cheek, cover the hole on the Yankauer, (if vented) then gently sweep over the arch of the tongue to the pouch of the opposite cheek.
6. Only apply suction through the Yankauer for a short period of time (no longer than 10 seconds) as this might result in tissue grab causing damage to the sensitive lining of the mouth. Repeat if necessary.
7. Administer oxygen pre and post procedure if indicated. This should be given at the prescribed flow rate for the patient.
8. Once the procedure has been completed, suction saline solution through the system.
9. After flushing the Yankauer and tubing with saline solution empty the suction canister by disposing of the contents appropriately.
10. If cleaned Yankauer is suitable for further use with the same patient within 24 hours, put back into plastic sheath to keep it clean.
11. Remove gloves (if worn) and wash hands with soap and water.

WARNING:

Do not resterilize, may result in infection or allergic reaction. Do not reuse, may result in inaccurate flow rate. Avoid freezing and excessive heat.

STORAGE TEMPERATURE:

-30°C/-22°F
+40°C/+104°F

For more info

 Telephone
| 323-761-0035

 E-mail
| info@rtghealth.com

 www.rtghealth.com