



REORDER NUMBER:

**RTG-04003**

**02-Life™**


**Oxygen Mask**

**Medium Concentration w/ 7' (2.1m) Tubing  
Universal Connector**

**Pediatric**

**RxOnly**

Caution: Federal law (USA)  
restricts this device to use by  
or at the direction of a physician

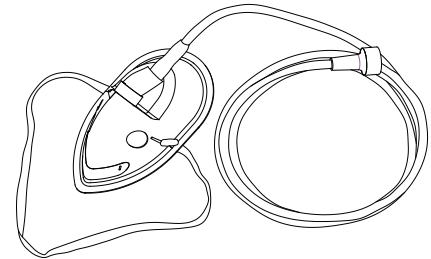

Non-Sterile



Not made  
with natural  
rubber latex



Do Not  
Re-use



Consult  
Instructions  
For Use

Clean, Ready to Use  
For Single Patient Use

## INSTRUCTIONS FOR USE:

### NOTE:

These directions are general guidelines intended for use by qualified medical personnel.

1. Attach oxygen supply tubing to oxygen source. Adjust flow as prescribed by physician.
2. Place mask over patient's face, covering both nose and mouth.
3. Slip elastic band over patient's head and adjust for comfort and fit.  
For best comfort, place straps below patient's ears.
4. Pinch nose clip to provide a seal around the nose.

## WARNING:

**WARNING: Re-processing of products intended for single use only may result in degraded performance or a loss of functionality.**

**WARNING: Re-use of single use only products may result in exposure to viral, bacterial, fungal, or prionic pathogens. Validated cleaning and sterilization methods and instructions for reprocessing to original specifications are not available for these products.**

**WARNING: This product is not designed to be cleaned, disinfected or re-sterilized.**

STORAGE TEMPERATURE:  
-30°C/-22°F  
+40°C/+104°F

For more info

 Telephone

| 323-761-0035

 E-mail

| info@rtghealth.com



| www.rtghealth.com