



REORDER NUMBER:

**RTG-06002**

**02-Life™**



**Oxygen Tubing**

**14' (4.3m) Crush Resistant Tubing**

**Standard Connector**

### RxOnly

Caution: Federal law (USA) restricts this device to use by or at the direction of a physician

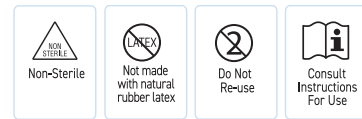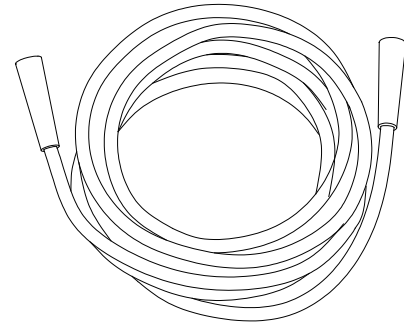

Clean, Ready to Use  
For Single Patient Use

### INSTRUCTIONS FOR USE:

#### NOTE:

These directions are general guidelines intended for use by qualified medical personnel.

1. Connect oxygen tubing between the device being used and the gas source.
2. Set the gas to the proper flow as prescribed by physician or licensed practitioner. Check for gas flow through the device.

### WARNING:

- WARNING:** Re-processing of products intended for single use only may result in degraded performance or a loss of functionality.
- WARNING:** Re-use of single use only products may result in exposure to viral, bacterial, fungal, or prionic pathogens. Validated cleaning and sterilization methods and instructions for reprocessing to original specifications are not available for these products.
- WARNING:** This product is not designed to be cleaned, disinfected or re-sterilized.

STORAGE TEMPERATURE:  
-30°C/-22°F  
+40°C/+104°F

For more info

Telephone

| 323-761-0035

E-mail

| info@rtghealth.com

| www.rtghealth.com