



REORDER NUMBER:

**RTG-07202**

**Adapt O-Life™**

**Aerosol Tee Adaptor**


**15mm I.D. x 15mm O.D.**

**Spring loaded**

**Valved**

## RxOnly

Caution: Federal law (USA) restricts this device to use by or at the direction of a physician

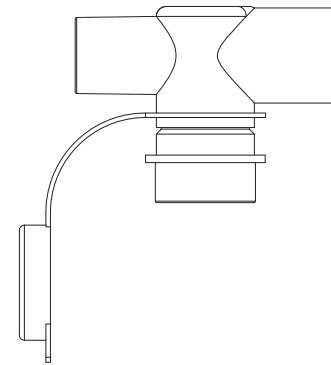

Clean, Ready to Use  
For Single Patient Use

## INSTRUCTIONS FOR USE:

### NOTE:

These directions are general guidelines intended for use by qualified medical personnel.

1. Place tee adaptor in desired location in corrugated tubing.
2. Maintain cap in closed position when not in use.
3. Open cap prior to using.
4. Activate the valve by placing the nebulizer into the base of the tee adaptor.
5. To ensure a tight fit, press and twist the nebulizer firmly into place.
6. Once procedure is completed, twist and pull to remove the nebulizer.
7. Reclose cap.

## WARNING:

**WARNING: Re-processing of products intended for single use only may result in degraded performance or a loss of functionality.**

**WARNING: Re-use of single use only products may result in exposure to viral, bacterial, fungal, or prionic pathogens. Validated cleaning and sterilization methods and instructions for reprocessing to original specifications are not available for these products.**

**WARNING: This product is not designed to be cleaned, disinfected or re-sterilized.**

STORAGE TEMPERATURE:  
-30°C/-22°F  
+40°C/+104°F

For more info

 Telephone

| 323-761-0035

 E-mail

| info@rtghealth.com



| www.rtghealth.com